

USING KLARO™ SURGICAL LIGHTING DEVICE




Retractor Loops

Take caution when opening to ensure accountability of 4 retractor loops provided.



1 Peel Open the Tyvek lid on the plastic blister tray. KLARO™ is supplied sterile.

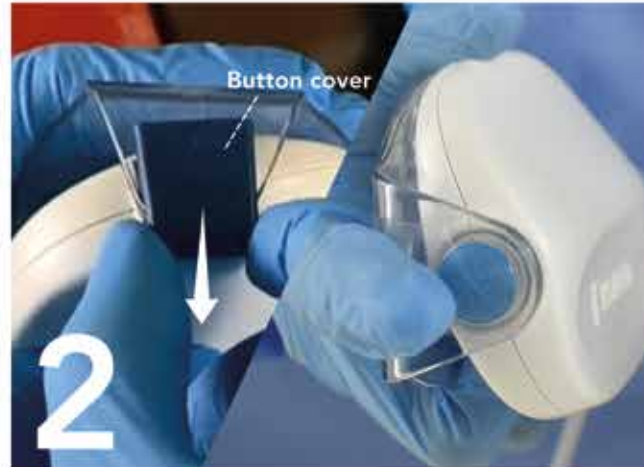

2 Remove the button cover and press on the round blue button to **switch KLARO™ on**. With each press the lighting intensity increases. (4 Lighting Intensity Levels)



3 Use the light strip as it is OR **bend** it to achieve wide-angled flood lighting.



KLARO™ Demo Video



4 Clip the battery pack to surgical drapes for hands-free operation.




Be alerted to 15-minutes of battery life remaining when the red light appears.

5

Dispose KLARO™ as medical waste. Leave KLARO™ on to drain off residual Lithium.*

*KLARO contains lithium-metal batteries (non-rechargeable).

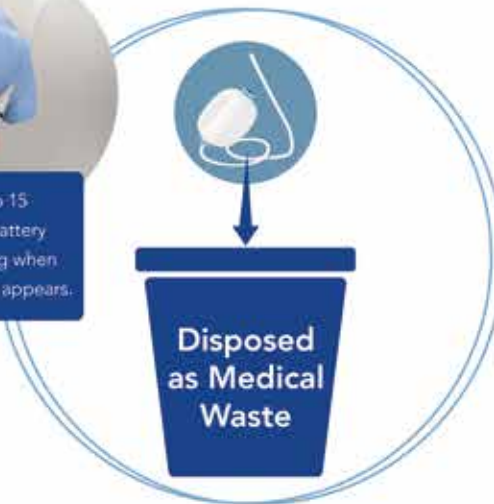

As a Standalone

Position the KLARO™ light strip within surgical cavities to achieve improved illumination and better visibility.



With Surgical Retractor

Mount the KLARO™ light strip onto handheld retractors using 2 retractor loops.

